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User Guide

DivaCup

Manufactured for Diva International Inc. 222 McIntyre Drive, Kitchener, ON Canada N2R 1E8 and distributed by Herberry Ltd. at 2-10-3 Kojimachi Chiyoda-ku Tokyo. Marketing Authorization Holder is Integro Ltd. at 5-3-1 Kachidoki Chuo-ku Tokyo.

Questions?

(P) +050-1744-4881

(E) info@herberry.jp

1. Soft silicone rim and cup

The DivaCup is made from durable silicone that becomes more flexible with body heat.

2. Leak-resistant air holes

The four holes under the rim of your DivaCup create a seal that secures the cup in place. Goodbye leaks.

3. Sturdy base

Easy to grip for easy removal. You shouldn't feel the stem at all, but it's there to help you find the base when you need it.

4. Trimmable stem

Just in case you want to take things down a notch, you can trim the stem to fit your body's comfort zone (just don't go shorter than ¼ inch, or 0.6 cm).

Sizing Chart

Model 0

New to periods, 18 years of age or younger.

0

Model 1

Between ages 19 and 30, with medium menstrual flow.

1

Model 2

Over the age of 30 and/or have a heavier menstrual flow.

2

For additional information, please visit our website: www.divacup.com.

Congratulations

Congratulations on choosing the DivaCup®! The revolutionary menstrual care product.

Prior to using your DivaCup for the first time please carefully read this user guide and keep it for future reference. It will help you use your DivaCup with confidence. If you have any gynecological, medical concerns or conditions please consult with your healthcare provider before using your DivaCup.

What is in the DivaCup box?

Inside the DivaCup box you will find your DivaCup, a storage bag and this user guide.

How to use your DivaCup

BEFORE YOU USE YOUR DIVACUP: Boil your cup for 5-10 minutes. Wash your hands thoroughly prior to insertion.

How do I insert the DivaCup?

1. FOLD THE CUP: The team at DivaCup have tested several methods and recommend two different fold options: the U-Fold and the Push Down fold. We recommend you try both fold options to see which one is right for you.

U-FOLD: Press the sides of the cup between your fingers and fold it in half to create a U.

OR


PUSH DOWN FOLD: With your index finger on the top rim of the cup, push the rim down into the center of the cup base to create a triangle.

Once you have mastered your folds you are ready to insert your DivaCup!

2. INSERTING THE DIVACUP: While sitting on the toilet or standing with one leg up on the bathtub, gently separate your labia with your free hand. Holding the folded cup between your thumb and forefinger, push the curved edge of the folded cup into your vagina horizontally.

Your DivaCup may open before it is fully inserted into your vagina. Don't worry, this is normal. Continue to gently push the cup into your vagina until the stem is even with the vaginal opening. Inserting the DivaCup too high may cause leaks and make removal difficult.

Tip: Aim the folded DivaCup toward your tailbone at the base of your spine, away from the cervix.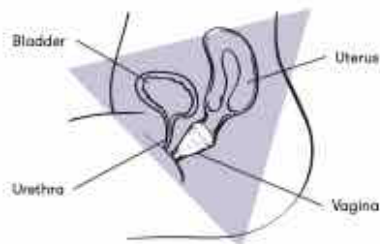

Rotate the cup: Once the DivaCup is in your vagina, grip the base of the cup, not the stem. Turn the cup one complete rotation, 360 degrees, in either direction. This will create a seal.

Tip: An alternative insertion method is to insert the cup part way into your vagina, rotate it 360 degrees, then push it horizontally the rest of the way. We recommend that first time users try using the DivaCup during the daytime.

To prevent leaking: To prevent leaking, once inserted, try sliding your finger along the outside of the cup and your vaginal wall, pressing gently on the sides of the cup. This should open the cup fully, creating the seal and will make rotation easier.

Trim the stem: Some DivaCup users find that the stem is too long and causes irritation. If the stem feels uncomfortable, remove the cup and trim the stem slightly, but no shorter than 1/4 inch (0.6 cm).

How long can I leave my DivaCup in? You can expect no leaks for up to 12 hours. Depending on your menstrual flow, you may have to empty it more frequently. It is recommended that you empty and clean your cup 2–3 times in a 24-hour period.

How do I remove my DivaCup? Before removing your DivaCup you will need clean hands and for those first-time users, some patience.

- 3. REMOVING THE DIVACUP:** Get comfortable, sit on the toilet, or stand with one leg on the bathtub and gently separate your labia with your fingers. With your forefinger and thumb, gently pull on the stem until you can feel the base of the cup. Pinch the base of the cup. This will break the seal. Gently move the cup from side to side and pull down. This action will remove the cup from your vagina.

Caution: Do not aggressively pull on the stem to remove the cup. This will be painful and you will likely spill the menstrual flow. **Never** use a foreign object, such as tweezers, to remove your DivaCup.

Once removed, keep your cup upright so that your flow remains in the cup. Empty the contents and thoroughly wash your DivaCup with warm water and a mild, unscented, oil-free soap. For best results, we recommend using the DivaWash®. Now your DivaCup is ready to reinsert.

I am having trouble removing the DivaCup, what do I do? Not to worry. Remember that your vagina is only 3–4 inches/8–10 cm long and the cervix will prevent your DivaCup from going anywhere else in your body.

If your DivaCup has been in for 12 hours or less try to relax. Relaxed muscles will help with removal. Turn on your favorite show, enjoy a nice cup of tea or snack, relax and let gravity do its work.

If you are still unable to remove your DivaCup after 12 hours, contact your healthcare provider for assistance with removing your DivaCup.

- 4. CLEANING AND CARE:** Just like your skin, your DivaCup deserves the best! Wash it thoroughly with warm water and a mild, unscented, oil-free soap. For best results, we recommend using the DivaWash.

Stay clear of home remedies such as vinegar, tea tree oil, castile soap, anti-bacterial soap, hand sanitizers, hydrogen peroxide or any other harsh chemicals.

Store the DivaCup in the cotton bag provided, not in a plastic bag or airtight container.

Tip: The tiny holes under the rim of your DivaCup must stay clean at all times. We recommend that during cleaning, you stretch the cup around the holes and run it under warm running water.

How do I know when to replace my DivaCup? We recommend that you replace your DivaCup annually. However, it's important that you discontinue using your DivaCup immediately if you notice a sticky or powdery film on the surface of the

cup, a foul odor from the cup, if it accidentally falls in the toilet, or you used it while you had a yeast or bacterial infection.

Anything else I should know about my DivaCup?

There are just a few extra things you should know before using your DivaCup:

- Follow these directions and only use your DivaCup during menstruation.
- Keep this user guide for future reference.
- Remove the DivaCup prior to having sex. The DivaCup is not a contraceptive device and will not protect you against pregnancy or sexually transmitted infections.
- If you wear an IUD we recommend you consult your healthcare provider prior to using the DivaCup to understand IUD benefits and risks.
- Do not use the DivaCup if you have a yeast or bacterial infection. Resume use of a DivaCup once the infection has completely cleared.
- Dispose of the DivaCup in the trash or recycle where silicone recycling facilities exist. Do not flush the DivaCup down the toilet.
- Toxic shock syndrome (TSS) is a rare, but serious disease caused by toxin-producing strains of the *Staphylococcus aureus* bacterium. In extremely rare cases, the strain may present itself with infection linked to the use of internal menstrual care products.

We recommend that you consult your healthcare provider prior to using the DivaCup if you have been previously diagnosed with TSS. Discontinue use of the DivaCup and contact your healthcare provider if you experience any of the following symptoms: sudden vomiting, diarrhea, high fever, headache, sunburn-like rash, muscle aches, confusion or seizures.

- If you experience an increase in back pain near your kidneys while wearing the DivaCup and the pain stops once the cup is removed we recommend you discontinue the use of the DivaCup and consult with your healthcare provider.
- Incorrect use of the DivaCup may result in difficulty removing the cup or minor complications. If you experience any adverse effects when using the DivaCup, discontinue use and consult your healthcare provider.
- Diva International Inc. cares about your health and safety. Menstrual cups are personal hygiene devices and may not be returned or exchanged. All sales are final.

THE DIVACUP IS MADE OF MEDICAL GRADE SILICONE
AND MANUFACTURED IN CANADA.

THE DIVACUP DOES NOT CONTAIN ANY ADDED CHEMICALS
(NO BPA), LATEX, DYES OR PLASTICS.

The DivaCup is intended to be used as a receptacle,
which is placed in the vagina to collect menstrual flow.

Notification medical device number: 13B3X10237HB0001

Hygiene products (01) Menstrual tampon | General medical device
Menstrual tampon | JMDN 35694000

UG-V1-EJ Dec 19