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IVA Cannula

General Information

Refer to the label on the product or package before use.

Indications for Use

The IVA Cannula is used with an endoscope to collect or inject tissues/cells and/or medicinal solution through the artificial opening during endoscopic treatment.

Contraindications

- Do not reuse. (Reusable for single patient only after resterilization)

Product Description

The product is inserted into channel of endoscope and the cannula tip is delivered to target site while observing with endoscope. For conducting collection of tissues/cells, aspirate into shaft using handle of inner shaft. For injecting medicinal solution, syringe is connected to the connector of shaft and it is injected from cannula tip.

Instructions for Use

Preparation

1. This product is sterilized. Do not reuse (The product is reusable for single patient only after resterilization).
2. Confirm endoscope to be used in advance.
3. Inspect the sterile primary package, appearance and function of the product. Do not use if any abnormality is noted.

Instructions for Use

1. Insert the product into channel of endoscope.
2. Move the cannula tip up to target site observing with endoscope. Collect and/or inject tissues/cells and/or medicinal solution and/or expand lumen.
3. Carefully and gently remove the product from the insertion site.

Reuse

1. The product is reusable for single patient only after resterilization. When reusing the product, inspect to ensure that the product has no damage, properly works and suitable for use. Wash, disinfect, sterilize the product adequately. Conduct the sterilization in reference with the following conditions.
Ethylene oxide gas concentration: $\geq 500\text{mg/L}$.
Sterilization condition: $40\text{--}55^{\circ}\text{C}$, ≥ 4 hours, 0.10Mpa , flushing ≥ 5 times
2. After sterilization, inspect the sterile primary package, appearance and function of the product. Do not use if any abnormality is noted.

Precautions

- Read the package insert prior to use.
- Use the product by operator having seasoned knowledge about procedure for the product.
- Do not use for patients unsuitable for use.
- Do not use after expiration date.
- Open the package immediately before use.
- Wear gloves to avoid such as infection.
- Do not use product with the broken package or any abnormalities of the product.
- Use the product aseptically. Carry out enough precautionary measures against infectious microorganism such as bacteria.
- Do not apply excessive force to the patient and the product. Confirm the product is usable for the procedure in advance.
- Do not make any modification to this product.
- Confirm the condition of patient during use. Cease using if abnormality is noted.
- When foreign body is attached in the channel of endoscope, difficulty for insertion cannula may be raised. Confirm condition of endoscope before using the product.
- The product is ethylene oxide gas-sterilized before opening primary package.
- This instruction is only suggestion. The final decision on the technique and procedure used is the responsibility of the user.

Storage Instructions and Expiration Date

Store in a clean and dry condition away from sunlight, heat and humidity. Expiration date is shown on the product label.

Symbols on the Label

	Do not reuse
	Batch code
	Re-order number
	Sterilized using ethylene oxide gas
	Use by